GENESIS 50 RFT

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Computer controlled highfrequency x-ray-generator

Radiography – Fluoroscopy – Digital Radiography

Radioagaphy

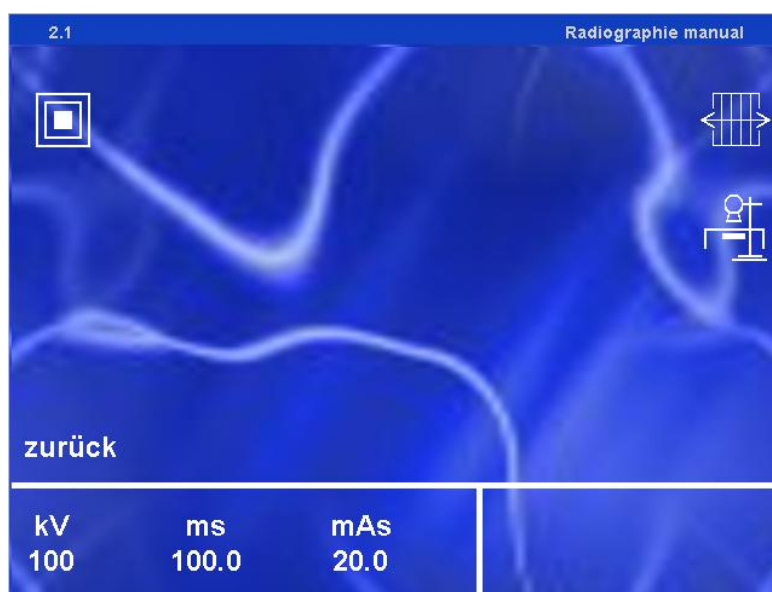
In the manual radiography mode, all the exposure parameters can be selected directly by the user.

The built in security circuits prevent the selection of exposure parameters which could destroy the generator and/or the tube.

The integrated Dose Area Measuring System shows the dose, the patient got during the exposure.

Also the automatic exposure control (AEC) can be used in the manual mode.

The after exposure display shows the most important exposure parameters (mAs, kV).

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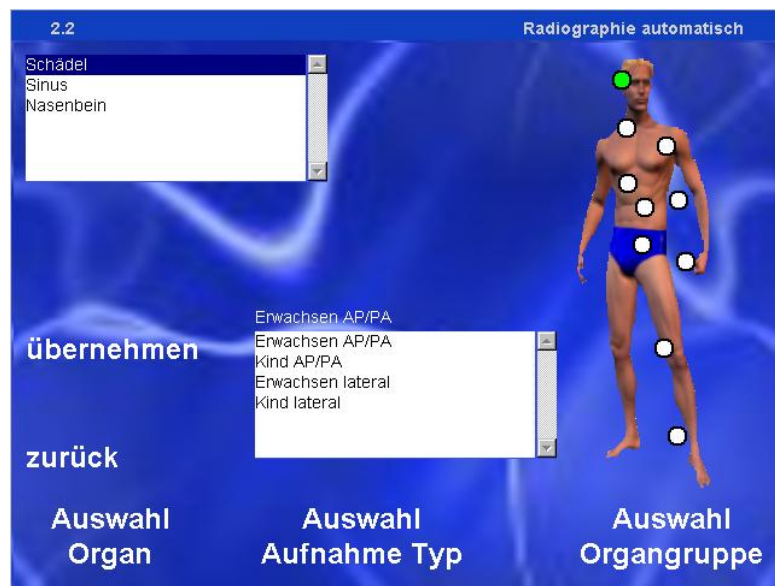
Programmable Anatomical selection

The GENESIS 50 RFT has a built in data base with a lot of anatomical selections.

This data base contains exposure parameters for a lot of exposure techniques. All the data can be changed easily by the user.

The simple modification of exposure parameters can be entered with the keypad of the GENESIS 50 RFT. In order to adjust a text, a computer keypad and a mouse can be attached.

The exposure parameters of the last exposure can be uploaded to another computer system for documentation and/or storage together with the patients personal data (RS 232 – DICOM).

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Fluoroscopy

The Fluoroscopy function of the GENESIS 50 RFT has several options:

- Continuous fluoroscopy
- Pulsed fluoroscopy
- High current fluoroscopy
- The fluoroscopy time is monitored by the GENESIS 50 RFT
- Automatic Bright Control (ABC)

Digital Radiography

The GENESIS 50 RFT is prepared to communicate with the newest digital radiography systems. The interface can be modified very easily to match the requirements of different systems. It is also possible to install the GENESIS 50 RFT together with a digital add-on system.

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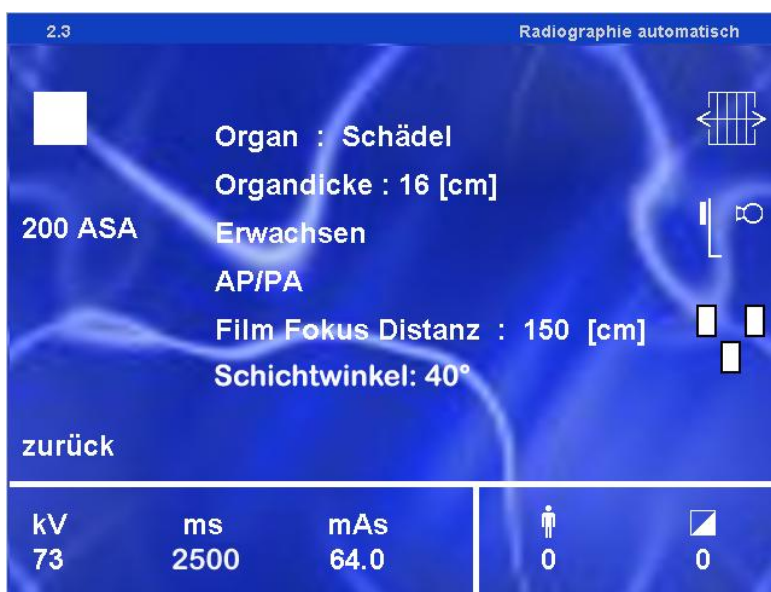
Tomography

The GENESIS 50 RFT is equipped with an interface to external tomography or planigraphy systems. The informations about the tomo-angle, the tomo-speed and the choosen mAs-product are evaluated by the GENESIS 50 RFT and the calculated tube current will be adjusted automatically.

The information stored in the anatomical data base can also be used in the tomography mode to make it easier to operate the GENESIS 50 RFT.

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